



2051 Junction Ave.
Suite #128, San Jose, CA 95131
1-800-889-9008

To: Valued Customers

January 2011

This memo is to inform you that our company sponsored an independent laboratory to validate Suggested Disinfecting Procedures for Bioteque pessaries. Under controlled laboratory practices, the result showed “no growth” using the following parameters.

Please note that many techniques are practiced in the health care community, and it is vital that every facility validate its equipment and parameters before processing any medical device.

Suggested Disinfecting Procedures for the Physician:

Cleaning: *submerge in enzymatic cleaner for 20 minutes*

Disinfecting: *use disinfecting solution like Cidex OPA for 12 minutes and rinse thoroughly*

Sterilizing: *30 minutes for 121 C Gravity Displacement Steam Cycles
4 minutes for 132-134 C Pre-vacuum Assisted Steam Cycles*

***Donut pessary cannot be sterilized in auto claver*

Gary Liu
VP in Regulatory Affairs

